


# Simple closing and locking instructions for your Sharpak Zero sharps container

Two closure positions, using our unique, patented VisiLOCK® lid closing technology:

**TEMPORARY** for when not in use. To temporarily close: rotate lid clockwise to the first closure position to completely enclose the contents. To open: rotate anti-clockwise

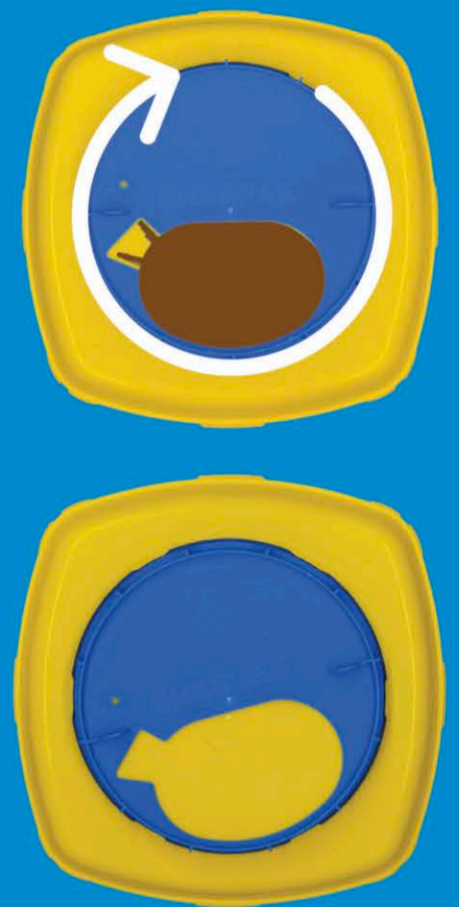

**PERMANENT** for when container is full and being disposed. To permanently close and lock: rotate lid clockwise past the first closure position. The spring-loaded red button will become visible with an audible click indicating the container is locked and ready for safe disposal



**VisiLOCK**

Remember, never leave the Sharpak Zero container unattended when the closure is open



Scan For Video

