

Simple closing and locking instructions for your Sharpak Zero sharps container

Two closure positions, using our unique, patented VisiLOCK® lid closing technology:

TEMPORARY for when not in use. To temporarily close: rotate lid clockwise to the first closure position to completely enclose the contents. To open: rotate anti-clockwise




PERMANENT for when container is full and being disposed. To permanently close and lock: rotate lid clockwise past the first closure position. The spring-loaded red button will become visible with an audible click indicating the container is locked and ready for safe disposal



VisiLOCK

Remember, never leave the Sharpak Zero container unattended when the closure is open

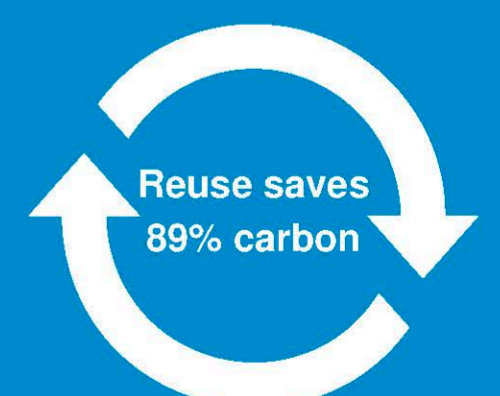

Scan For Video

